

## MIDI DESCEMET INSERTER

IT IS A NEW DESCEMET LENTICLE **INSERTION SYSTEM FOR DMEK** PROCEDURES FOLLOWING OUR FIRST **DEVICE LAUNCHED YEARS AGO.** INNOVATION IS PART OF OUR DNA, IT **ENABLES US TO RESPOND TO THE NEEDS OF** A MARKET IN CONTINUOUS EVOLUTION. **OUR STRENGTH AND PERSISTENCE HAS ALWAYS GUIDED US WITH AN INNOVATIVE SPIRIT. THE RESULTS WE HAVE ACHIEVED HAVE ALLOWED US** TO WIN THE TRUST OF OUR **CUSTOMERS IN INTERNATIONAL** V. Samicola M.O MARKETS.



THE SYSTEM IS INTRODUCED WITH AN INNOVATIVE ERGONOMICS, WHICH FACILITATES THE INTRUDUCTION OF THE GLASS TIP IN THE ANTERIOR CHAMBER. THE ANGLE, SIMILAR TO MANY OTHER TRADITIONAL SURGICAL INSTRUMENTS, ALLOWS A MORE NATURAL AND PHYSIOLOGICAL APPROACH.


IN THE FIRST STEP THE CENTRAL GLASS CAPTURES THE ROLL (DM), THROUGH A GENTLE AND CONTROLLED SUCTION, AN **INNOVATIVE FILTER SYSTEM** CONTROL PREVENTS THE GRAFT FROM BEING SUCKED INTO THE SYRINGE.

IN THE SECOND STEP, THE "MIDI" CARTRIDGE (MICRO INCISION DESCEMET INSERTER), IS MOUNTED ON THE MAIN CONTAINER MAKING THE SYSTEM READY TO INJECT THE ROLL IN A SIMPLE, PROPER AND SAFE WAY.

IMPORTANTLY, THE MAIN

**CONTAINER GLASS CAN BE ROTATED** 

THUS ALLOWING PROPER
ORIENTATION OF THE GRAFT
BEFORE INJECTING

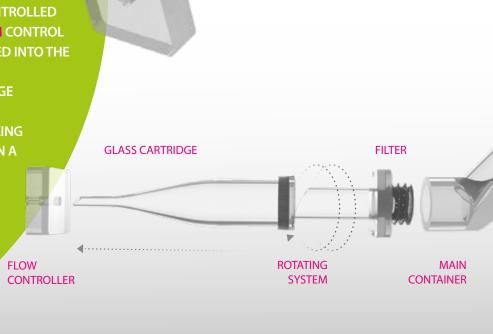





Sistema di UNI EN ISO 9001 Qualità Certificato UNI CEI EN ISO 13485 Certified © Copyright: Quality System Hermann Janach



Via Borgo Vico, 35 - 22100 Como - Italy Tel. +39 031 574088 - fax +39 031 572055 international@janach.it